

▲ Open here

▲ Open here


Polyp Collection Kit REF BBP-50-S

【Main Material】 PC, PVC, HDPE

【Indications】

To collect the polyps through the endoscope channel which are greater than 1mm, for pathological analysis.

【Structure】

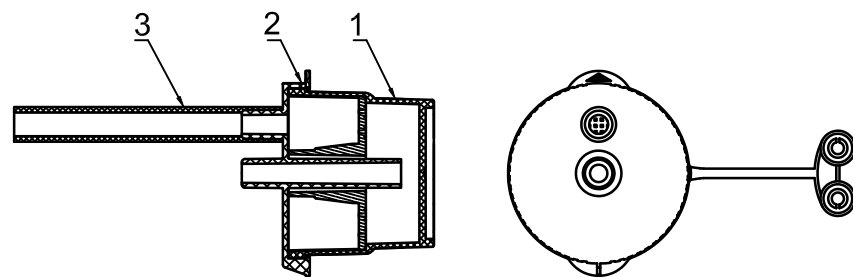

1.Bottle 2.Lid 3.Collection hose

【User Instruction】

1.Operation

- 1.1. Connect the lid to the suction unit.
- 1.2. Connect the collection hose to the endoscope.
- 1.3. To entrap the polyp into the bottle.

2.Removal

Open the lid, remove the tissue specimen by using tweezers and other tools from the bottle.

【Notes】

- 1.Operator must be trained and qualified
- 2.The product is disposable for one patient



1. "LOT NUMBER" SEE SEALING PART
2. SHELF LIFE: 5 YEARS

Rx only



Manufacturer

Micro-Tech (Nanjing) Co., Ltd.

No.10 Gaoke Third Road, Nanjing National Hi-Tech Industrial

Development Zone, Nanjing 210032, Jiangsu Province, PRC

TEL:+ 86 25 58609879, 58646393

FAX:+ 86 25 58744269

Email: info@micro-tech.com.cn

www.micro-tech.com.cn



MICRO-TECHTM
ENDOSCOPY

Distributed By

Micro-Tech USA Inc.

2855 Boardwalk Drive, Ann Arbor, MI 48104 USA

Tel: 734-259-3768

Toll free: 877-552-4027

info@micro-tech-usa.com



Shanghai International Holding Corp. GmbH (Europe)

Eiffestrasse 80, 20537 Hamburg, Germany